# Latex-Free Transducer Covers

CIV-Flex™ • NeoGuard™ • iVAS™ • Poly • Surgi™ • Surgi-Tip™

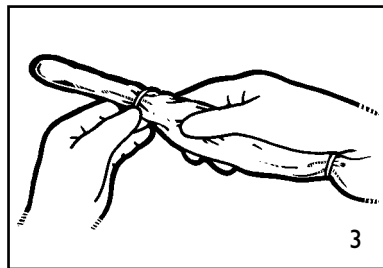
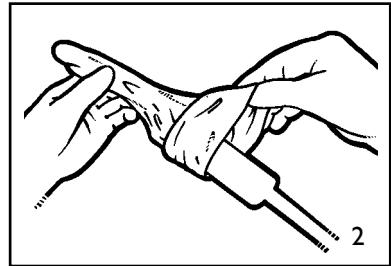
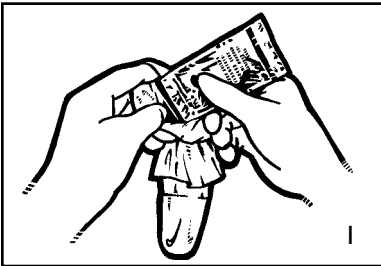
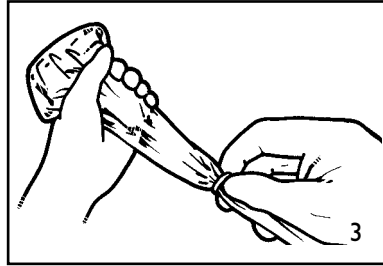
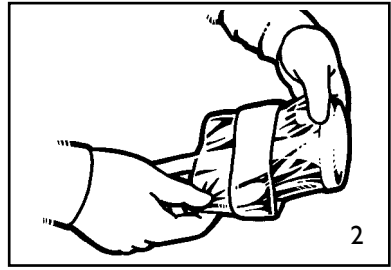
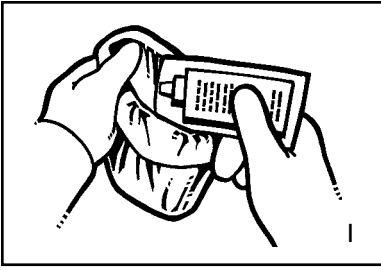
---

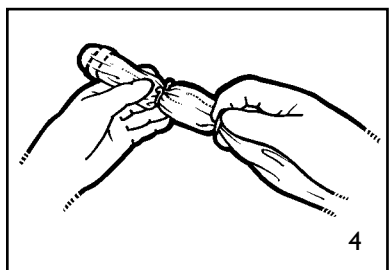
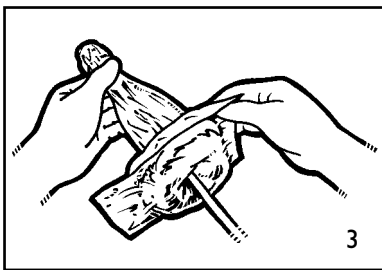
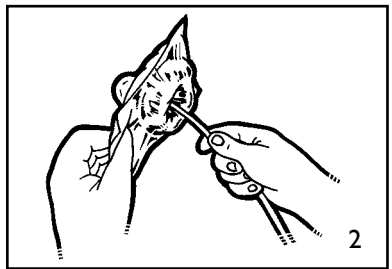
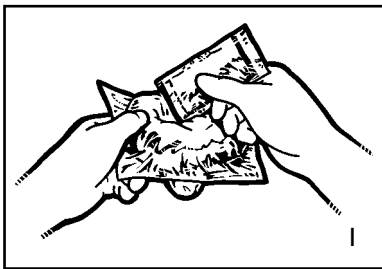
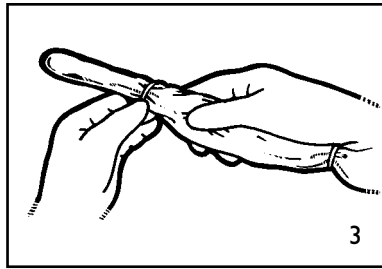
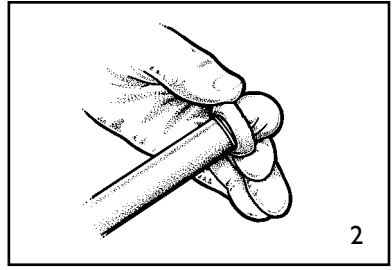
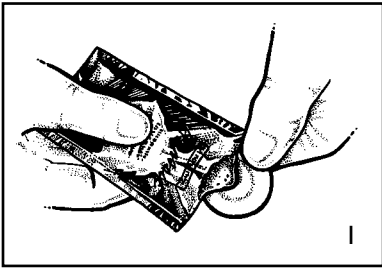
Reference Guide

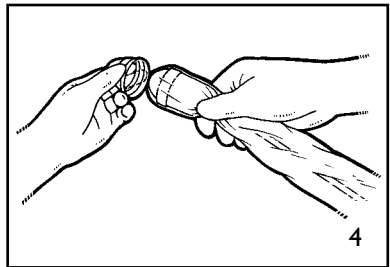
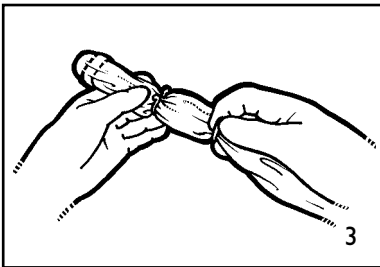
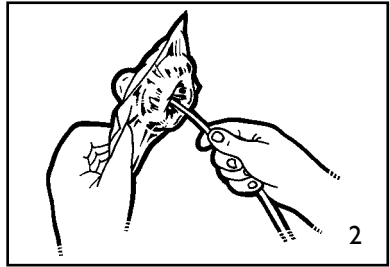
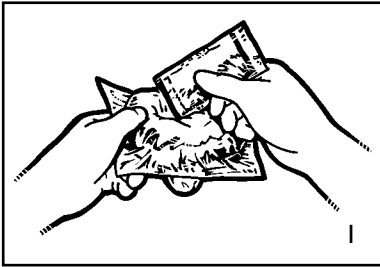
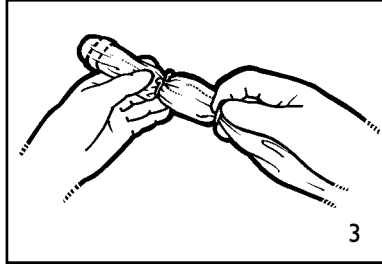
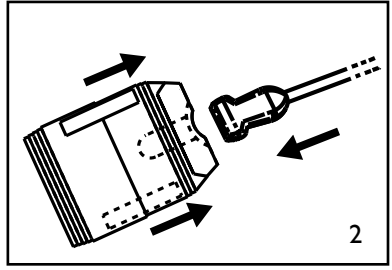
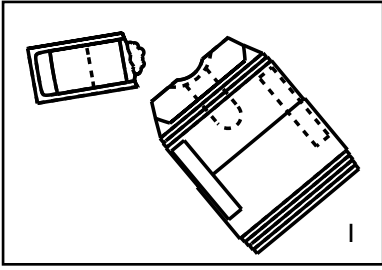
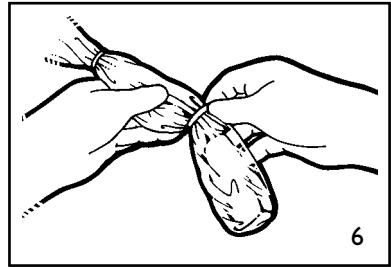
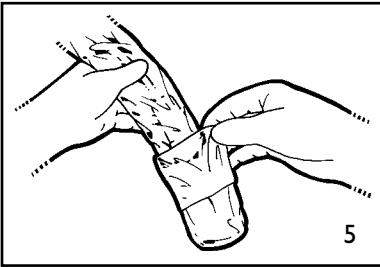
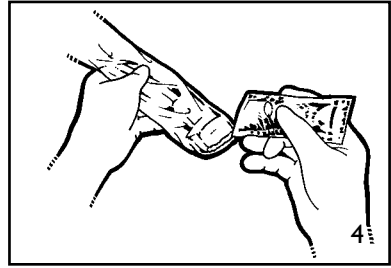
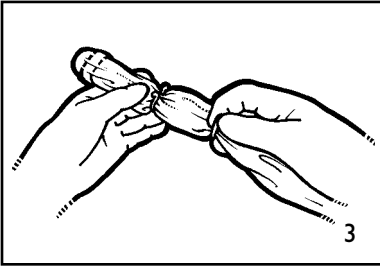
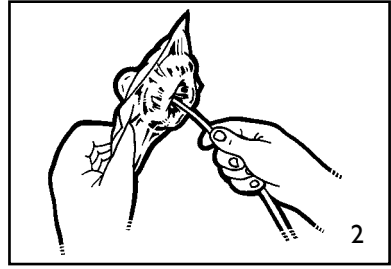
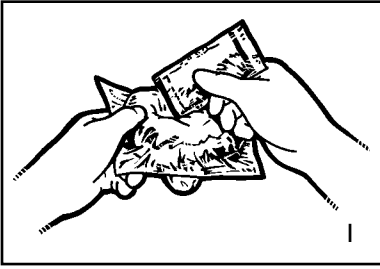
CE  
0120

**INTENDED USE**

- The cover allows use of the transducer in scanning and needle guided procedures for body surface, endocavity and intra-operative diagnostic ultrasound, while helping to prevent transfer of microorganisms, body fluids and particulate material to the patient and health care worker during reuse of the transducer.

**WARNING**

- *In the USA, federal law restricts device to sale or use by or on order of a physician.*
- *Before use, you should be trained in ultrasonography. For instructions on the use of your transducer, see your system's manual.*
- *Dispose of single-use components as infectious waste.*
- *See your system's manual for reprocessing transducer between use.*

**Covering the Transducer and Bracket**

**WARNING**

- *Use only water-soluble agents or gels. Petroleum or mineral oil based materials may harm cover.*
- *Disposable components are single-use only. Do not use if expiration date has passed.*
- *For disposable components labeled sterile, do not use if integrity of package is violated.*
- *For illustration purposes only, transducer may be shown without a transducer cover. Always place a cover over transducer to protect patients and users from cross-contamination.*

1. Place an appropriate amount of gel inside cover and/or on transducer face. Poor imaging may result if no gel is used.
2. Insert transducer into cover making sure to use proper sterile technique. Pull cover tightly over transducer face to remove wrinkles and air bubbles, taking care to avoid puncturing cover.
3. Secure cover with enclosed bands (if applicable).
4. Inspect cover to ensure there are no holes or tears.

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---

**Corporate Headquarters:** 102 First Street South, Kalona, IA 52247-9589 USA  
319.656.4447, 800.445.6741 | fax: 319.656.4451, 877.329.2482  
[www.civco.com](http://www.civco.com)

COPYRIGHT © 2011 ALL RIGHTS RESERVED. CIVCO IS A REGISTERED TRADEMARK OF CIVCO MEDICAL SOLUTIONS. CIVIFLEX, IVAS AND NEGUARD ARE TRADEMARKS OF CIVCO MEDICAL SOLUTIONS. ALL OTHER TRADEMARKS ARE PROPERTY OF THEIR RESPECTIVE OWNERS. ALL PRODUCTS MAY NOT BE LICENSED IN ACCORDANCE WITH CANADIAN LAW. PRINTED IN USA.

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MPS GmbH