# Cord Covers & System Drapes

CIV-Flex™ • Poly

---

REFERENCE GUIDE

CE  
0120

**INTENDED USE**

- *The cover allows use of the transducer in scanning and needle guided procedures for body surface, endocavity and intra-operative diagnostic ultrasound, while helping to prevent transfer of microorganisms, body fluids and particulate material to the patient and health care worker during reuse of the transducer.*

**WARNING**

- *In the USA, federal law restricts device to sale or use by or on order of a physician.*
- *Before use, you should be trained in ultrasonography. For instructions on the use of your transducer, see your system's manual.*
- *Disposable covers and drapes are packaged sterile and are single-use only. Do not use if integrity of packaging is violated or if expiration date has passed.*
- *Dispose of single-use components as infectious waste.*
- *Users of this product have an obligation and responsibility to provide the highest degree of infection control to patients, co-workers and themselves. To avoid cross-contamination, follow infection control policies established by your facility.*
- *See your system's manual for reprocessing transducer between use.*

**COVERING THE CORD** \_\_\_\_\_

1. Using proper sterile technique, hold cord cover so entire length does not unfold, possibly causing contamination.
2. Insert distal end of cord into cover and proceed toward proximal connection, sliding cover to desired position.
3. Secure cover with enclosed bands.
4. Inspect cover to ensure there are no holes or tears.

**COVERING THE SYSTEM** \_\_\_\_\_

1. Using proper sterile technique, hold system drape so entire length does not unfold, possibly causing contamination.
2. Remove backing from adhesive strips and secure drape to control panel.
3. Inspect drape to ensure there are no holes or tears.

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COPYRIGHT © 2010 ALL RIGHTS RESERVED. CIVCO IS A REGISTERED TRADEMARK OF CIVCO MEDICAL SOLUTIONS. ALL OTHER TRADEMARKS ARE PROPERTY OF THEIR RESPECTIVE OWNERS. PRINTED IN USA. ALL PRODUCTS MAY NOT BE LICENSED IN ACCORDANCE WITH CANADIAN LAW.
MPS GmbH